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User Manual for Industrial Scale integrated with
Newster technologies

Industrial Scale User Manual

newster® **NW5**

newster® **NW15**

newster® **NW30**

newster® **NW50**

newster® **NW50S**

Index

1. General Information	3
1.1. Preamble.....	4
2. Use.....	5
2.1. Location.....	6
2.2. Registration of the weight of the waste to be sterilized (CER 18.01.03).....	6
2.3. Registration of the weight of the sterilized waste (CER 20.03.01).....	7

1. GENERAL INFORMATION

1.1. Preamble

The present user manual is for those Newster System srl clients that have purchased one or more NW5-NW15-NW50 sterilizers for the treatment of infectious healthcare waste and contains operative indications for the integration and use of the "Cogo Bilance" industrial scale. Infact for a correct management of potentially infectious healthcare waste it is important to register the kg of waste in entrance to the sterilizer and also kg in exit at the end of the cycle. The present user manual illustrates the various steps of the registration of the weight of the waste to be sterilized, the subtraction of the weight of the collection bin containing the waste and the weight of the sterilized waste.

Proper use

The "Cogo Bilance" industrial scale is to be considered "not automatic", meaning that the waste to be weighed in its container is placed with caution by hand in the middle of the scale plate. The weighing value can be read after the weight indication has stabilized.

2. USE

2.1. Location

- The “Cogo Scale” connected to the Newster sterilizer provides reliable weight reading only if the following rules are respected :
- The scale must be placed on a flat and stable area;
- Avoid impact during weighing phase;
- The scale must be protected from the eventual presence of vapor and dust

2.2. Registration of the weight of the waste to be sterilized (CER 18.01.03)

Placed the bin containing the potentially infectious medical waste to be treated on the scale.

- STEPS

Press the scale's plus button  to sum the “kg weight” of the waste to be sterilized . Repeat the above operation with each bin until you reach the kg of potentially infectious waste necessary for each cycle. The total kg of potentially infectious medical waste varies according to the Newster sterilizer model (NW5-NW15-NW50) connected to the “Cogo Scale”.

The total weight is indicated under “total”. Manually indicate the total number of bins weighed.

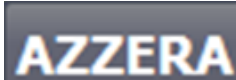
After weighing the bins push the button  to register the pre-treatment data


-SUBTRACTION OF THE WEIGHT OF WASTE TO BE STERILIZED

When necessary, push the minus button  to deduct "kg weight" from "scale weight".


- TO ZERO THE WEIGHT OF THE WASTE TO BE STERILIZED

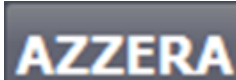When necessary, push the zero button  to zero " kg weight", "scale weight" and the "total".

2.3. Registration of the weight of the sterilized waste (CER 20.03.01)

Press the green and red buttons at the same time for at least two seconds in order to zero the kg weighed.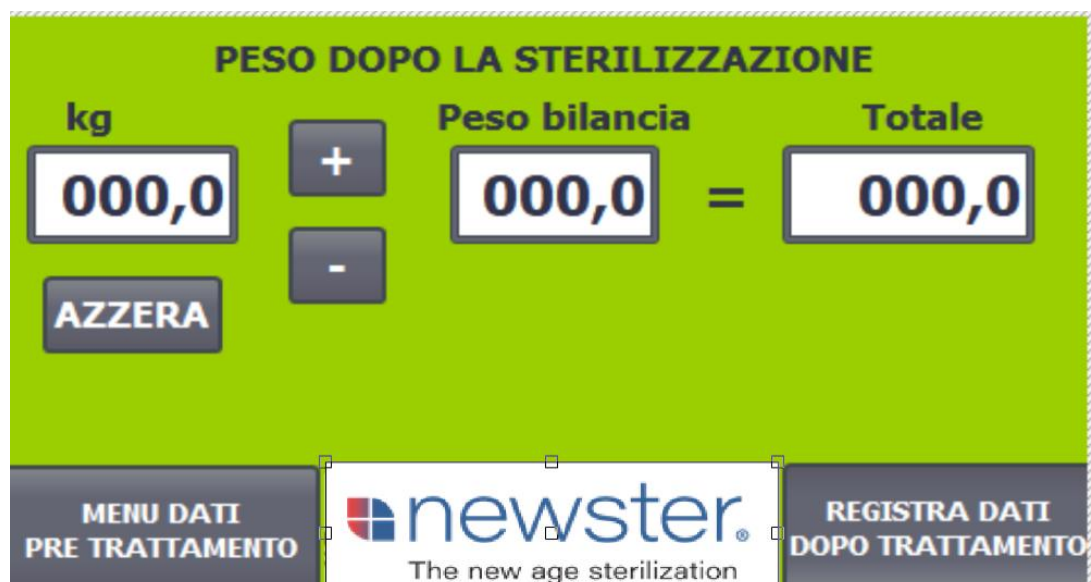

- SUM OF THE WEIGHT OF THE WASTE TO BE STERILIZED

Press the plus button  to sum the "Kg weight" of the sterilized waste with the "scale weight" at the end of every sterilization cycle. The total weight is indicated in the "total".


After the weighing press the  post treatment register button.

- SUBTRACTION OF THE WEIGHT OF WASTE STERILIZED

When necessary, push the minus button  **to deduct** "kg weight" from "scale weight".

- TO ZERO THE WEIGHT OF THE WASTE STERILIZED

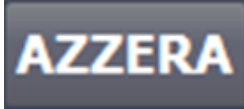When necessary, push the zero button  **to zero** " kg weight", "scale weight" and the "total".