---

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---

User manual for bin washer sanitizer integrated  
with Newster Sterilizers

# User Manual for bin washer sanitizer

newster® **BWS6**

newster® **BWS8**

newster® **BWS13**

newster® **BWS15**

---

## Index

1.	general information .....	3
1.1.	Preface .....	4
1.2.	Use of the machine .....	4
2.	Instructions for the user .....	5
2.1.	Specifications and location of use .....	6
2.2.	Access to the bin washer sanitizer Menu from the NW operator panel .....	6
2.3.	Graphic interface of bin washer sanitizer integrated to the NW sterilizer .....	7
2.4.	Bin washer sanitizer operating parameters.....	7
2.5.	Errors and Messages .....	9

## **1. GENERAL INFORMATION**

## **1.1. Preface**

The present user manual contains operating instructions for the usage of plastic bin washer sanitizer integrated with NW5-NW15-NW50 sterilizers for hospital waste.

In fact, in order to have a proper management of the potentially infectious solid waste, it is also important to wash and sanitize the plastic bins.

Therefore, the present manual aims at briefly illustrate the different phases related to start-up of the machine, the operator interface panel and the related error messages.

## **1.2. Use of the machine**

The machine is designed to wash and sanitize plastic bins generally used for the collection of potentially infectious solid waste.

The machine does not contain any automatic loading system. This means that the bin, including its lid, must be manually placed inside the washing-sanitizing chamber.

## **2. INSTRUCTIONS FOR THE USER**

## 2.1. Specifications and location of use

- The bin washer sanitizer integrated with NW sterilizers guarantees washing and sanitization;
- Easy and flexible to use;
- Self-cleaning;
- Programmable working parameters;
- Low energy and water consumption;
- Low maintenance;
- Maximum reliability;
- The bin washer sanitizer must be located on a stable and flat surface.

## 2.2. Access to the bin washer sanitizer Menu from the NW operator panel

In order to access the bin washer sanitizer menu press the button

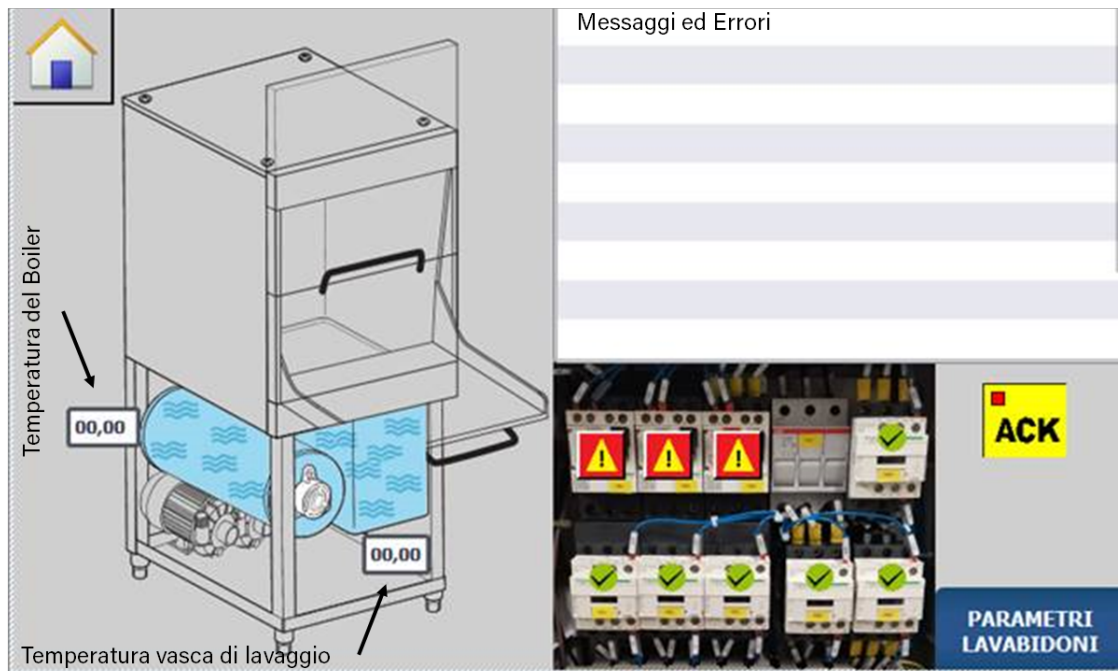
located on the NW operator panel.

### 2.3. Graphic interface of bin washer sanitizer integrated to the NW sterilizer

The graphic interface of the bin washer sanitizer contains the main following sections:

- a) Errors and messages;
- b) Boiler and washing tank temperature;
- c) Button to access the bin washer sanitizer operating parameters

### 2.4. Bin washer sanitizer operating parameters

In order to access the bin washer sanitizer operating parameters press the

button .

The operating parameters section contains:

- 1) Washing tank temperature;
- 2) Boiler temperature;
- 3) Additional operating parameters.

The minimum boiler temperature required to start the washing and sanitizing

cycle  must be equal and not lower than 60 °C.

The minimum washing tank temperature required to start the cycle

 must be equal and not lower than 35°C.

The maximum temperature required to stop the boiler electrical resistances

 must be equal to 80 °C.

The maximum temperature to stop the electrical washing tank resistances

 must be equal to 50 °C.

The minimum temperature to start the boiler electrical resistances

 must be equal to 75 °C.

The minimum temperature to start the washing tank electrical resistances

 must be equal to 45 °C.

## 2.5. Errors and Messages

The main errors and messages are listed below:

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“Wash pump motor protection alarm QM1” indicates that the wash pump protection magnetothermic has tripped;

“Vapor suction motor protection alarm QM2” indicates that the suction fan protection magnetothermic has tripped;

Alarm motor protection rinse pump QM3" indicates that the rinse pum protection magnetothermic has tripped;

"Bin washer sanitizer door open" indicates that bin washer sanitizer door is opened, preventing to start the sanitizing cycle;

"Boiler thermostat Safety Alarm" indicates a too high boiler temperature;

"The boiler pressure or rinsing pump broken" indicates the rupture of one of the two components;

"Tank pressure broken" indicates that the pressure switch is broken;

"Low boiler water level" indicates that the boiler water level is too low, preventing to start the sanitizing cycle;

"Low tank water level" indicates that the tank water level is too low, preventing to start the sanitizing cycle;

"Tank water temperature too low" indicates that the tank water temperature is too low, preventing to start the sanitizing cycle;

"Boiler water temperature too low" indicates that the boiler water temperature is too low, preventing to start the sanitizing cycle;

"non-selected cycle wash" indicates that the knob is positioned in the neutral position;

"No general machine start" indicates the failure of Newster machine;

"Cycle in progress" indicates that the sanitizing cycle is ON;

"OK, the cycle can starts" indicates that there are no malfunctions and / or alarm messages and the cycle can begin.